

EAHP 2018.

This document will assist you with viewing the eSlides for the upcoming EAHP conference.

To view the slides, you will need to:


1. Navigate to <https://www.slidehosting.eu/Login.php>

2. Login using the below credentials:


Username: EAHP2018

Password: EAHPconf!23

3. If you see the below screen, please click on 'Continue'

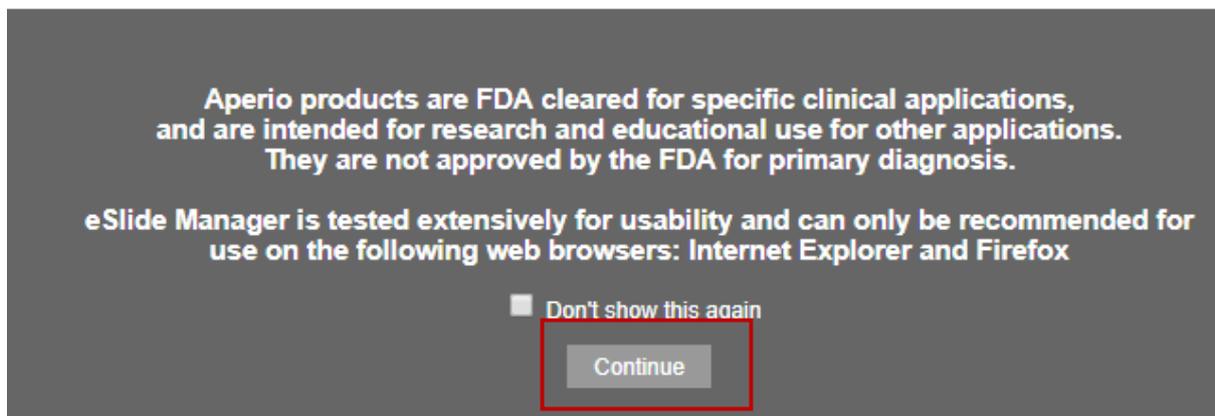

4. At the following screen please click on the 'Cases' Icon:

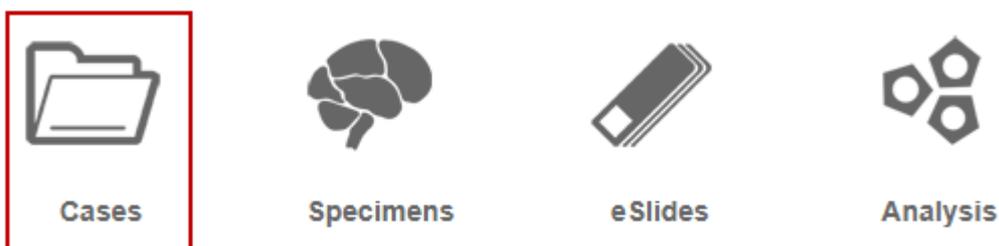

5. Please click on either the folder icon or the ID '876', as highlighted below:




6. You will now see a list of the slide groups, please choose the required group by clicking on either the Brain icon or on the ID number as shown below:

	<u>Procedure</u>	<u>Id ↓</u>	<u>Data Group</u>
	Slides 500 - 599	985	EAHP2018DG
	Slides 400 - 499	984	EAHP2018DG
	Slides 300 - 399	983	EAHP2018DG
	Slides 200 - 299	982	EAHP2018DG
	Slides 100 -199	981	EAHP2018DG

7. This will take you to the slides list. From here you can view the slide by clicking on the thumbnail of the image or if you want to see the attachments with the slide you will need to click on the slide icon. Both procedures are described below:
- To just view the slide please click on the Image thumbnail or Label thumbnail. The image will then be opened in either your browser or if you have ImageScope installed into this application.

Specimen eSlides


[Sort](#) | [View Images](#) | [Open Data](#) | [Analyze](#) | [Unassign](#) | [Annotations](#)
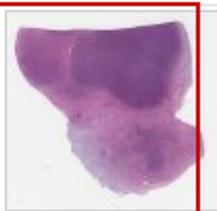
<input type="checkbox"/>		Label	Image
<input type="checkbox"/>			
<input type="checkbox"/>			

b. To open the attachments please click on the following icon of the required slide:

Specimen eSlides


[Sort](#) | [View Images](#) | [Open Data](#) | [Analyze](#) | [Unassign](#) | [Annotations](#)
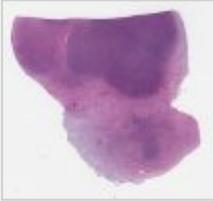
<input type="checkbox"/>		Label	Image
<input type="checkbox"/>		 LYWS-100	
<input type="checkbox"/>		 EAHP 18 LYWS-103	

c. Scroll down to the bottom of the screen and click on the plus sign next to the title 'eSlide Attachments', as shown below:

eSlide Attachments

eSlide Images

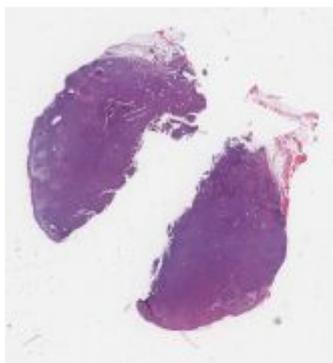

[Delete Image](#)

Description:


Interpretation:

Image ID: [3910](#)

File Location: [\\virdu-aws-sh2\images\EAHP2018\100-199\LYWS-103.svs](#)

d. Please click on the required file to start the download, as shown below:

+ eSlide Attachments
- eSlide Images

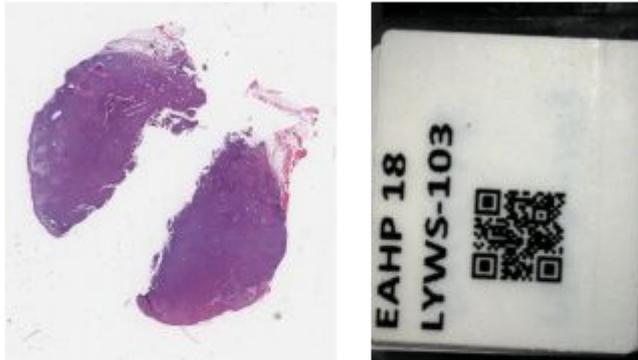

[Delete Image](#)

Description:
 Interpretation:
 Image ID: [3910](#)
 File Location: \\virdu-aws-sh2\images\EAHP2018\100-199\LYWS-103.svs

e. This file will then download to your machine and you can open this from the downloads folder.

8. If you want to continue on to a different slide, then you can do any of the following:
 a. In the current screen please click on either 'Previous eSlide' or 'Next eSlide':

eSlide Details

[< Previous eSlide](#)
[Next eSlide >](#)

[eSlide List](#) | [Assign To Existing Specimen](#) | [Assign To New Case](#) | [Assign To Existing Case](#)

Stain:
 Other:
 Comment:

Data Group: EAHP2018DG

eSlide Attachments

[Sort](#) | [Add Attachment](#) | [Delete](#) | Page **1** of 1 (2) ([Previous](#) | [Next](#))

<input type="checkbox"/>	Name ↑	Description	Tasks
<input type="checkbox"/>	EAHP18-LYWS-103.pdf	PDF	Edit
<input type="checkbox"/>	File upload_6010870.pdf	PDF	Edit

Display Records Per Page

eSlide Images

[Delete Image](#)

Description:
 Interpretation:
 Image ID: [3910](#)
 File Location: \\virdu-aws-sh2\images\EAHP2018\100-199\LYWS-103.svs

- b. If you want to look at a different eSlide within the current set of eSlides, please click on 'View from Specimen Level' and work from point 7.

▣ **Specimen Details**

[View from Specimen Level](#) | [Assign To New Case](#) | [Assign To Existing Case](#)

Procedure:	Slides 100 -199
Id:	981
Data Group:	EAHP2018DG
Body Site:	
Comment:	

- c. If you want to look at a different set of slides, please click on 'View from Case Level' and work from point 6.

▣ **Case Details**

[View from Case Level](#)

Id:	876
Volume:	EAHP2018
Pages:	
Month:	
Year:	
Author:	
Title:	
Comment:	EAHP Conference 2018
Patient Summary:	
Date Reported:	
Status:	
Case Acc#:	
Gender:	
Data Group:	EAHP2018DG
	eSlideShare

Appendix A.

If there is an attachment for an eSlide that was not submitted you can view these from the 'Specimens' window, at this screen please click on the plus symbol next to the title 'Specimen Attachments':

Specimen Details

[< Previous Specimen](#) | [Specimen List](#) | [Assign To New Case](#) | [Assign To Existing Case](#)

Procedure:	<input type="text" value="Slides 100 -199"/>
Id:	<input type="text" value="981"/>
Data Group:	<input type="text" value="EAHP2018DG"/>
Body Site:	<input type="text"/>
Comment:	<input type="text"/>

Specimen Attachments

Specimen Gross Images

And click on the desired attachment:

Specimen Attachments

[Sort](#) | [Add Attachment](#) | [Delete](#) Page of 1 (6) ([Previous](#) | [Next](#))

<input type="checkbox"/>	Name ↑	Description	Tasks
<input type="checkbox"/>	EAHP18-LYWS-155.pdf	Slide 155 PDF	Edit
<input type="checkbox"/>	EAHP18-LYWS-168.pdf	Slide 168 PDF	Edit
<input type="checkbox"/>	EAHP18-LYWS-172.pdf	Slide 172 PDF	Edit
<input type="checkbox"/>	File upload_6030374.pptx	Slide 172 Powerpoint	Edit
<input type="checkbox"/>	File upload_6031566.pptx	Slide 155 Powerpoint	Edit
<input type="checkbox"/>	File upload_6045980.pptx	Slide 168 Powerpoint	Edit

Display Records Per Page

This file will then be downloaded to your machine and can then be opened from your downloads folder.